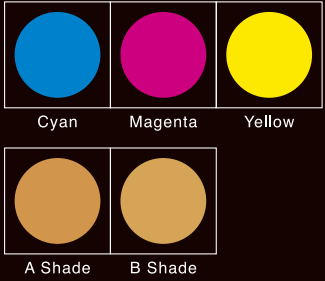
## Shade Adjustment

A and B shades of color paste are done to match Classical shades. They are used for the adjustment of chroma of tooth color restorations.  
 ●After applying shade stain, the surface should be covered by translucent and so on.

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
## LITE ART Color Line-up

Shades of LITE ART (total 15 colors) are selected based on the popular shades in clinical use from SHOFU porcelain stain "VINTAGE Art" and are also originally created with distinctive features of organic pigments.

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## What's CMY Color?

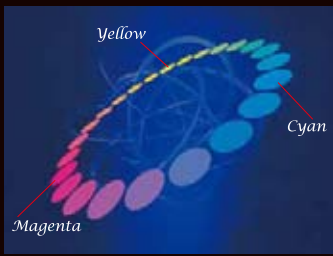Red, Yellow and Blue are generally considered to be the three primary colors as they were also stated in classical study of chromatics. However, if they were mixed with each other, there were differences between the mixing ratio and the image of color to be created. And the range of coloration was also limited. Currently, in various fields such as color printing, a subtractive color mixing theory of three primary colors (Cyan, Magenta and Yellow) proposed by Ducos du Hauron in 1868, is used and, as a result, matching of color mixing ratio and image of color to be created became possible, and the range of individual coloration was also expanded.

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# Recipe for VINTAGE Art Color using LITE ART CMY Color

Shades of VINTAGE Art can be created by mixing Cyan, Magenta, Yellow, White and Black of LITE ART.

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- \*The Yellow color of LITE ART differs somewhat from the Yellow color of VINTAGE Art.
- The colors in gray-shaded region in the mixing ratio are available in LITE ART system.
- The printed colors might be different from that of actual colors in this system.

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