


How to update the firmware of EyeSpecial C-V

1. Verify the current firmware

- ① Power the camera on.
- ② Verify the current firmware version on start-up screen.

Current firmware version is shown here.

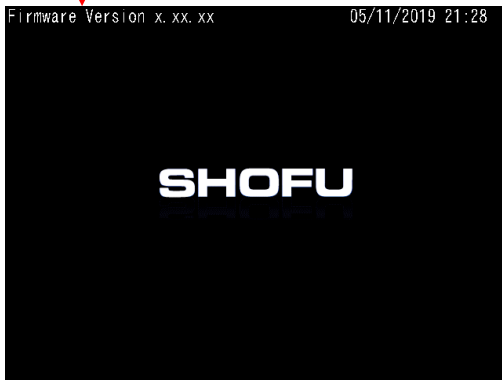

※The latest version of firmware is available at Shofu website.

(https://www.shofu.co.jp/eyespecialc1/firmware_en.html)

※If your displayed current firmware matches the latest version, no further action is required.

- ③ Power the camera off.

2. Download the firmware

Download the latest version of firmware from Shofu website.

@j • HD , , É @ ~ È [È D ^ ^ •] ^ & ã & Í È { , æ ^ ^ } È |


3. Copy the firmware to a blank SD card



Attention!


- Do not use SD cards with Wi-Fi capabilities such as Eye-Fi card.
- The original SD card provided with your EyeSpecial C-V is recommended for the update.
- SD card should be blank (excepted for the firmware only).

- ① Delete all the data in the SD card.
- ② Copy the latest firmware (UPDATE5.FRM) to the blank SD card.

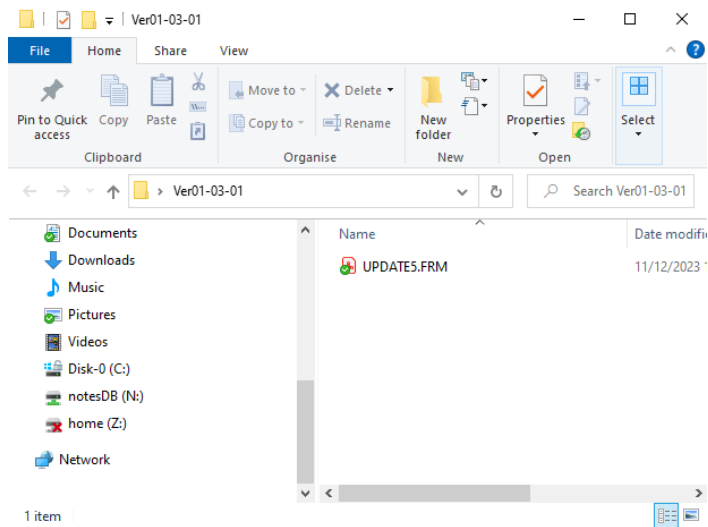

Reference

- If the SD card is unlocked, the firmware (UPDATE5.FRM) in the SD card is automatically deleted after the update.
- When updating more than one unit at a time, lock the SD card before the update to avoid deletion.

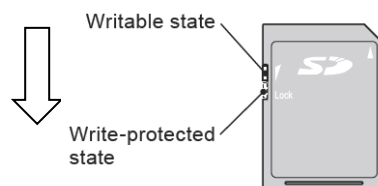


4. Update the firmware




Attention!

- Replace with new alkaline batteries or fully-charged chargeable batteries before updating the firmware.
- While updating the firmware:
 - Never power the camera off or remove the batteries.
 - Never remove the SD card.
 - Do not press any keys or buttons and rotate the dial.


- ① Insert SD card contains only the firmware into the SD card slot while the camera is powered off.
- ② Power the camera on.
- ③ Touch "OK".

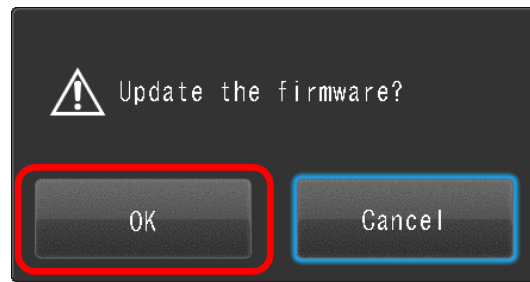

- ④ Wait until the update is completed.

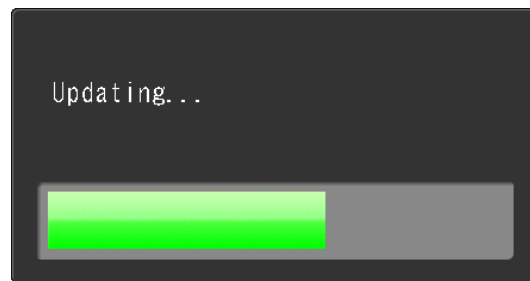

- ⑤ When the update is completed, the screen shows the message below. Confirm the firmware version shown is the latest version.

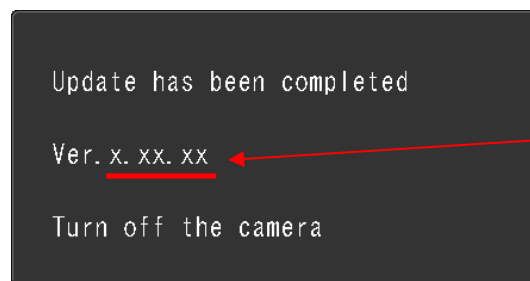

Updated firmware version

- ⑥ Power the camera off.
- ⑦ Eject the SD card and verify if the current firmware version is the latest version according to step 1.



Attention!


- **If the SD card is unlocked during the update process, all data including the firmware will automatically be permanently deleted.**

5. Delete the firmware in the SD card

When updating while the SD card is locked, delete the firmware in the SD card after the update by either method below.

[Delete the firmware by formatting in the camera]


- ① Slide the write-protect switch to unlocked position.

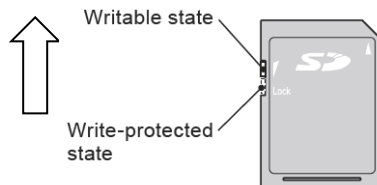

- ② Insert the SD card into the camera.
- ③ Format the SD card according to page 41 of User's Manual.

[Delete the firmware on the PC]

- ① Slide the write-protect switch to unlocked position.

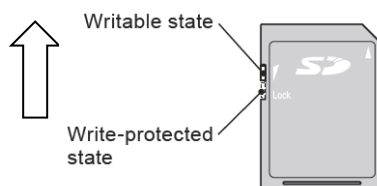

- ② Insert the SD card into the PC.
- ③ Delete the firmware (UPDATE5.FRM).